## **ATTENTION**

There is a potential risk of serious injury or death associated with virtually every product regardless of its type or intended use. Stander, Inc., has worked tirelessly to minimize all risks associated with its mobility aids, yet given the types of products, users, and a myriad of things outside of our control, we cannot eliminate every conceivable risk. Accordingly, a consumer must use common sense when utilizing our products, and take into account the user's overall physical and mental health.

Read all of the materials that accompany this product before it is installed or used.

## **LIMITED WARRANTY**

Congratulations on your purchase of your Stander's brand product. We hope that it will ease your daily living. Please read the following warranty statement for your protection.

Stander, Inc. warrants to the original purchaser that this product will be free from defects in material and/or workmanship for 1 year from the date of purchase. During the warranty period we will correct any defects in material or workmanship at no charge. All shipping cost must be paid by the original purchaser. Any replacement parts will be new and are warranted for the remainder of the original warranty or thirty (30) days from the date of shipment, whichever is longer. The original purchaser must notify Stander, Inc. during the warranty period in writing of the defect in workmanship or material. This warranty is not transferable and covers only normal use. Stander will not refund or pay for any shipping charges. The warranty does NOT cover: (1) Damage during shipment other than original shipment to retailer; (2) Damage caused by unauthorized attachments, alterations, or modifications; (3) Damage caused by the use of the product for purposes other than those for which the product was designed or intended; and (4) Damage caused by any other abuse, misuse, mishandling or misapplication.

Under no circumstances shall Stander, Inc. be liable for any special, incidental, or consequential damages based upon breach of this limited warranty, breach of contract, or strict liability.

THERE ARE NO WARRANTIES WHICH EXTEND BEYOND THOSE INDICATED HEREIN. ALL OTHER WARRANTIES, EXPRESS OR IMPLIED (INCLUDING ALL IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE), ARE HEREBY DISCLAIMED.

Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This warranty gives you specific legal right, and you may also have other rights which vary from state to state. To obtain service under this warranty, you can reach Stander, Inc. at (800) 506-9901 or (435) 755-0453.

#### **WARNINGS**

- ❖ It is critical that all users and installers of this product read and understand all of the materials provided with this product before it is installed or used. These warnings, as well as the assembly instructions, must be carefully followed to ensure that the product is properly installed and used. Failure to install or use this product properly could result in serious injury to the user(s). If there are any questions regarding the installation or use, associated risks or anything at all regarding this product do not install or use it and contact a Stander representative at (800) 506-9901 or (435) 755-0453. We are thrilled that you have chosen to use our product so we are more than happy to answer all of your questions.
- Individuals who have problems with memory, sleeping, incontinence, pain, uncontrolled body movement, who walk unsafely without assistance, or who are frail, weak, confused, restless, under the influence of drugs or any substance that could impact their balance or judgment are at a higher risk. You should first consult a physician before using this product.
- Improper installation of this product significantly increases the risk of injury.
- This product should not be used on any light weight furniture that would allow the product to become dislodged or move under during use of the product.
- Individuals who have size or weight conditions are at a higher risk of injury due to personal mobility issues and the ability of the product to be supported under the furniture.
- Children should not be allowed to hang on or play with this product.
- This product is not intended to carry the full weight of an individual. This product is only intended to provide balance and support while sitting and standing.
- This product may shift its position over time. Inspect this product frequently to ensure it is installed properly.



# **Instructions:**

1. Remove all components from the box and verify all parts are included as shown in Figure 1.


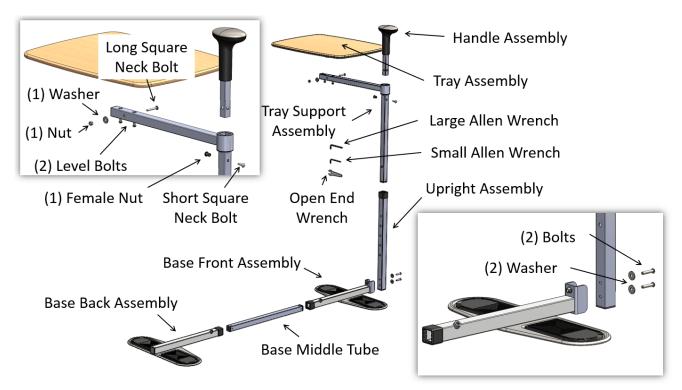


FIGURE 1

2. Slide <u>Base Front Assembly</u> and <u>Base Back Assembly</u> onto the <u>Base Middle Tube</u> as shown in Figure 2.

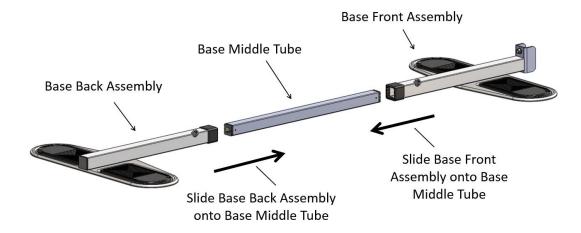

# FIGURE 2

3. Position the <u>Rubber foot pads</u> at the same distance of the <u>Furniture legs</u> spacing and tighten the <u>Set Screws</u> with the <u>Large Allen Wrench</u> as shown in Figure 3. The Furniture legs should sit on the center of the Rubber foot pads.

NOTE: The <u>Base Middle Tube</u> should engage the <u>Base Front Assembly</u> and the <u>Base Back Assembly</u> approximately 4 inches on each side in order for the Set Screws to properly engage.

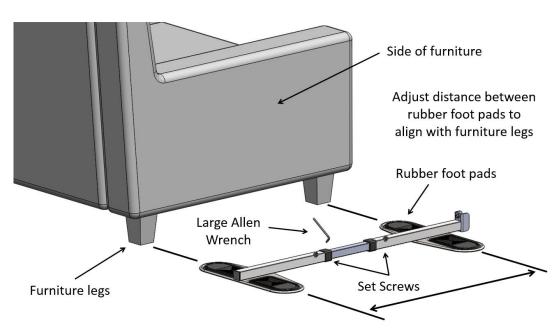


FIGURE 3

4. Attach the <u>Upright Assembly</u> to the <u>Base Front Assembly</u> with the <u>Small Allen Wrench</u>, <u>2 Bolts</u>, and <u>2 Washers</u> as shown in Figure 4.

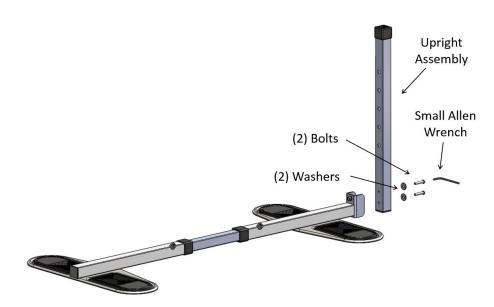


FIGURE 4

5. Using the Open End Wrench, attach the <u>Tray Assembly</u> to the <u>Tray Support Assembly</u> with the <u>Long Square Neck Bolt</u>, 1 <u>Washer</u>, and 1 <u>Nut</u> as shown in FIGURE 5.

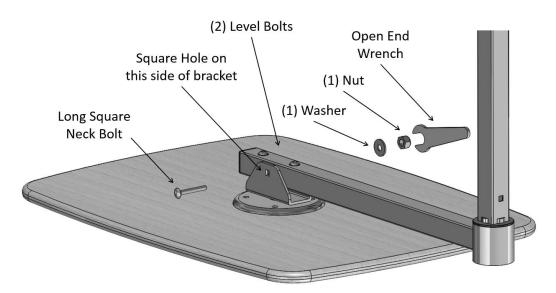


FIGURE 5

6. Depress the Spring Pin in the <u>Tray Support Assembly</u> and slide the <u>Tray Support Assembly</u> into the <u>Upright Assembly</u>. Allow the Spring Pin to engage any of the holes in the <u>Upright Assembly</u> as shown in FIGURE 6.

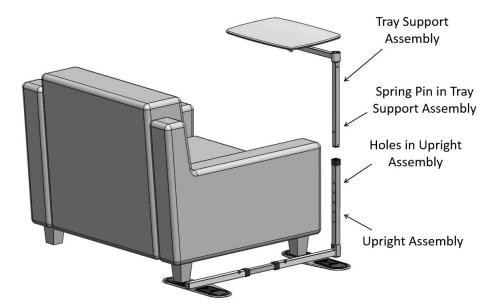


FIGURE 6

7. Adjust the Height of the <u>Tray Support Assembly</u> to pivot above the Furniture arm rest. **NOTE:** The Furniture legs can be positioned on either set of the Rubber foot pads as shown in FIGURE 7 and FIGURE 8.

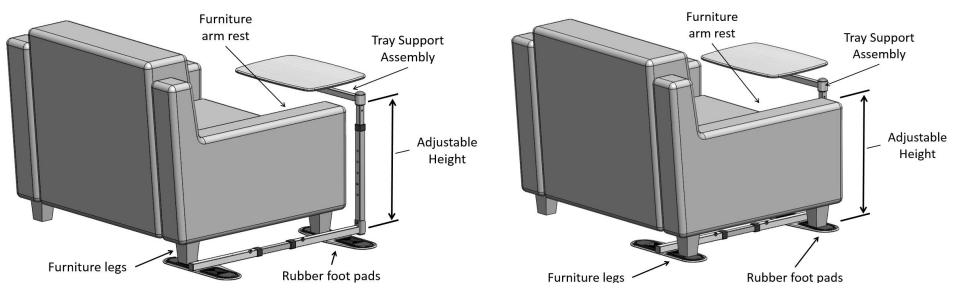


FIGURE 7 FIGURE 8

8. Attach the <u>Handle Assembly</u> to the <u>Tray Support Assembly</u> with the <u>Small Allen Wrench</u>, <u>Female Nut</u>, <u>Short Square Neck Bolt</u> as shown in Figure 9. **NOTE:** Short Square Neck Bolt must be inserted into the square hole of the <u>Tray Support Assembly</u>.

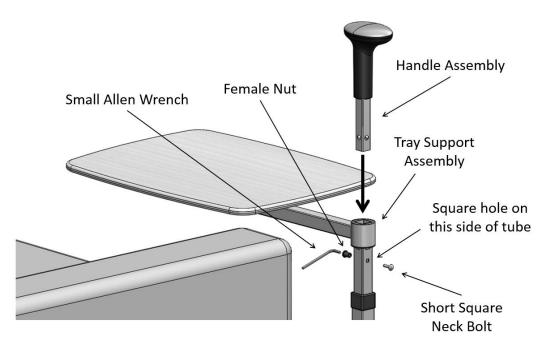


FIGURE 9

9. The <u>Tray Assembly</u> can be leveled by using the <u>Small Allen Wrench</u>. Loosen the leveling screw on the high side and then tighten the screw on the low side. When the <u>Tray Assembly</u> is level, tighten each screw equally as shown in FIGURE 10.

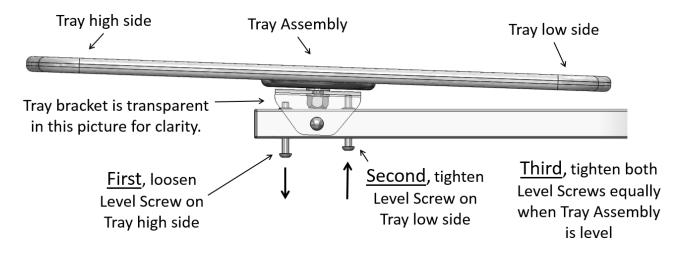


FIGURE 10