HANDY HANDLE™

SAFETY INFORMATION

The Handy Handle is designed to safely assist a person weighing up to 350 pounds. The only function of the Handy Handle is to assist in pulling a person to a standing position from a chair, bed, bench or other standard seat. The Handy Handle should never be used to pull a person up while sitting on the floor or the ground. The Handy Handle should never be used to pull someone up who is lying on the floor or the ground. Both persons using this product must have sufficient mobility, arm, and grip strength. Both persons using this product must have complete traction between both feet and the floor. Maintain grip until fully standing to prevent falling or injury. Do not use Handy Handles with anyone who cannot stand on their own without assistance. Please explain these warnings to anyone planning on using the product. Suitable to clean with common household cleaners or alcohol (70% Isopropanol).

DIRECTIONS

1. Both persons grasp the Handy Handle with both hands.
2. Move to the edge of the chair, bed, bench or standard seat.
3. While holding firmly, the sitting person should lean forward.
4. Both person's feet should have traction with the ground and say "ready, set, go" and begin standing on the word "go".

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