## UNIVERSAL STANDING HANDLE™

### Safety Information

The Handy Handle is designed to safely assist a person weighing up to 350 pounds. The only function of the Handy Handle is to assist in pulling a person to a standing position from a chair, bed, bench or other standard seat. The Handy Handle should never be used to pull a person up while sitting on the floor or the ground. The Handy Handle should never be used to pull someone up who is lying on the floor or the ground. Both persons using this product must have sufficient mobility, arm, and grip strength. Both persons using this product must have complete traction between both feet and the floor. Maintain grip until fully standing to prevent falling or injury. Handy Handles must not be used near dangerous objects including, but not limited to sharp or fragile objects. Do not use Handy Handles with anyone who cannot stand on their own without assistance. Please explain these warnings to anyone planning on using the product. Suitable to clean with common household cleaners or alcohol (70% Isopropanol).

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### Directions

1. Both persons using the Handy Handle grasp the grips with both hands.
2. Move to the edge of the chair, bed, bench or standard seat.
3. While holding the Handy Handle firmly, the sitting person should lean forward.
4. Make sure that both person's feet have traction with the ground.
5. Both persons using the handle should say "ready, set, go" and begin standing on the word "go."

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