

Standers™

Risers-345


User's Manual

PARTS LIST:

<u>ITEM</u>	<u>QTY.</u>	<u>DESCRIPTION</u>
1	8	Foam Pad
2	4	3" Riser
3	4	4" Riser
4	8	Non-Skid Rubber Strips

INSTRUCTIONS:

1. Center and apply adhesive backed Foam Pad (Item-1) to recess areas (A) and (B) of (Item-2) and (Item-3).
2. This product can be used 4 different ways! If you want to adjust the height of your bed or couch an additional 3", just place four 3" risers (Item-2) under the feet of your furniture. If you want to adjust the height of your bed or couch an additional 4", use only the four 4" risers (Item-3). If you want to adjust your bed or couch an additional 5", stack the 3" Riser (Item-2) on top of the 4" Riser (Item-3). Finally, you can use the 3" Riser (Item-2) and the 4" Riser (Item-3) separately on both your bed and couch!
3. **WARNING:** Do NOT mix 3" Risers (Item-2) with 4" Risers (Item-3). This will result in tipping of furniture and uneven balance of furniture weight, which may result in serious injury to the user(s), and/or damage to your furniture.
4. **WARNING:** Always place Risers directly under and centered on furniture feet. Make sure furniture feet rest on Foam Pads and NOT on the outer rim or the Risers! Failure to do this may cause the furniture to slip off the Risers and result in serious injury to the user(s).
5. **WARNING:** Non-Skid Rubber Strips (Item-4) have been provided to prevent the Risers from slipping on slick floor coverings such as wood, tile, and any other similar floor covering. On some carpeted floors, the Non-Skid Rubber Strips (Item-4) are not needed.
6. **WARNING:** DO NOT let children jump or play on your furniture. Improper use of the Risers may result in serious injury to user(s) and/or damage to your furniture.
7. Check your furniture from time to time, to make sure that the Risers are properly centered under the feet of you furniture.

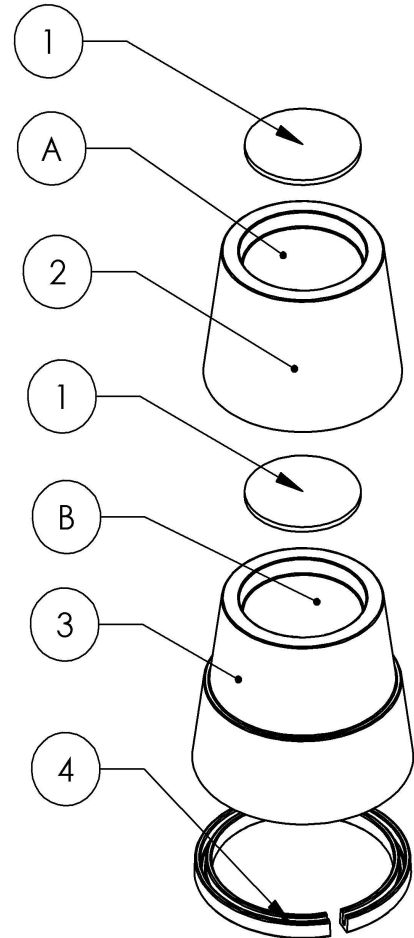

Visit our Website at:
www.stander.com

for new products like
the Bedcane, Couchcane,
and the Car Caddie!

Or call us at:

1-800-506-9901

for a FREE brochure of
our product line!